



Pee Cup

User Manual

Cat.#: PUC-50

HOW DOES IT WORK?

Premom Pee Cup is a sample collection tool designed to make urine collection easier for women. It is intended to be used with the Premom Fertility Ovulation Test and Pregnancy Test.

PREPARATIONS


- Ensure you have a flat surface nearby to place your cup on after collecting the sample.
- Wash hands thoroughly before collecting urine sample.
- Wash and dry the Pee Cup completely to remove all residue, or it may affect the test results.

HOW TO USE


1. Fully expand the bottom of the Pee Cup. Wash it with warm water and mild detergent before first use.

2. Thoroughly dry the cup and hold it by the handle.

3. Position the pee cup to collect your urine until it is about half full. Remove the cup and finish urinating into the toilet.

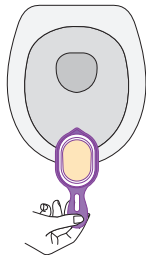

4. Dip your testing material (e.g., Ovulation or Pregnancy Test) into the urine for the duration specified in its instruction guide.

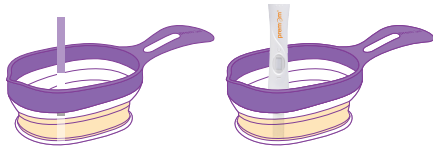


5. Pour your sample after reading the results. Wash thoroughly with warm water and mild detergent for at least 30 seconds, making sure to rub all areas to remove any residue.

WARNINGS AND PRECAUTIONS

- Use the Pee Cup only for its intended purpose of collecting urine samples. Do not use it for eating, drinking, or any other purpose.
- Do not share the Pee Cup for safety and hygienic reasons.
- Do not use the Pee Cup if it arrives damaged or becomes damaged in any way.
- Keep the Pee Cup out of reach of children; it is not a toy.
- Pee can naturally stain or discolor the Pee Cup over time.
- DO NOT place the Pee Cup in dishwasher.

PRODUCT COMPONENT

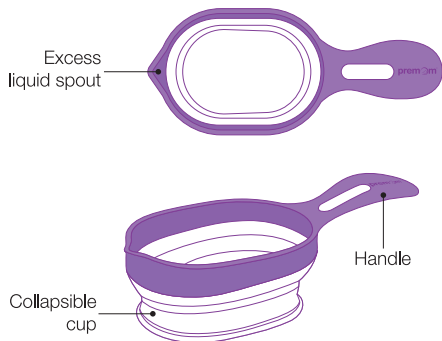

CONTENTS

- Pee Cup(s)
- User Manual

STORAGE AND STABILITY

Keep dry.

Keep away from direct sunlight.

Store in a clean, dry place at room temperature between uses.

QUESTIONS OR COMMENTS?

Please call toll-free at

1-855-822-6999

Monday-Friday 9:00 a.m.-5:00 p.m, Central Time.

support@premom.com

To learn more, please visit us at

www.premom.com

Manufactured for

Easy Healthcare Corporation

360 Shore Dr., Burr Ridge, IL USA 60527

Made in China