

Q1: How could I upload my BBT data to the App?

We recommend that you pair the thermometer to the Premom App before every measurement. To sync the temperature data to your account, enable the Bluetooth on your phone, open the Premom app, and turn on the thermometer. Once the thermometer is connected with the app, all the stored data will automatically sync to the app.

Q2: How could I get the previous temperature to display?

Memory stores up to 60 measurements on the device. Alternatively, the bluetooth setting will sync your BBT data automatically with unlimited measurements to the premom App via the cloud. While the thermometer is off, press down the On/ Off button for 3 seconds to review the stored data measurements in memory on the device. Also, you can track your BBT data on the Premom App.

Q3: Does the special sensor tip on this model work differently from the other basal thermometers? Is it waterproof?

This digital bbt thermometer provides a flexible light weighted thermometer oral sensor tip. Simply keep lying down and measure slightly under your tongue while not even holding on with your hand. This sensor tip is waterproof (but not the whole device) so it would be easily cleaned.

Q4. Is the beeping very noisy? Will it light up with a backlit when reading the measurement? I don't want it to wake my husband up every morning.

It has quiet micro beeps. It's very much like a digital watch chime in volume. If it was under a pillow you wouldn't hear it. In addition, it has a large backlit LCD display that helps you read easily in the dark.

Q5. Does this basal thermometer measure in Celsius?

Yes, our smart thermometer measures the temperature in Celsius as well as Fahrenheit.